

DOWNLOAD INSTRUCTIONS SOFTWARE UPGRADE v2.02

FOR GAIT TRAINER™ 3 & RTM600

950-400
950-402
950-406
950-420
950-422
950-426

The screenshot shows the Biodes website interface. At the top, there is a navigation bar with links for Physical Medicine, Nuclear Medicine, Medical Imaging, About Us, Support, and Contact Us. Below this is a breadcrumb trail: Home / Physical Medicine / Products / Software Updates / Gait Trainer Software Upgrade v2.02. The main content area features a large image of a computer monitor displaying the 'GAIT TRAINER 3' software interface. To the right of the image are buttons for 'Request Information' and 'Request a Quote'. Below the image, there is a 'Software' tab and a 'Manuals' tab. The text below the tabs states: 'Biodes has made recent upgrades to the Gait Trainer software, available to system owners.' Below this text is an 'Instructions:' section with the text: 'For download instructions, please click on the manuals tab above.' and a 'download' button. The right sidebar contains contact information: '1-800-224-6339' and 'INT'l 631-924-9000', along with buttons for 'Request Information' and 'Request a Quote'. There is also a 'View Catalog' link and a 'SENIOR REHAB' section with a 'Read More' button. The left sidebar lists various product categories under 'Physical Medicine', including Balance, Dynamometers, Ergometer, Treadmills, Semi-Reclining Elliptical, Supported Ambulation, Partial Body-Weight Support, Harnesses, and Software Updates. The Software Updates section lists: Biodes Concussion Manager Software, Balance System SD Software Upgrade v3.01, BiWay Software Upgrade v3.01, Gait Trainer Software Upgrade v2.01, and RTM600 Software Upgrade v2.01.

BIODEX

Biodes Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (Int'l 631-924-9000), Fax: 631-924-9338, Email: info@biodes.com, www.biodes.com

This manual covers download procedures for the following software:

Software Upgrade v2.02 for Biodex Gait Trainer™ 3 and/or RTM600

This software can be used with the following products:

Version 2.02 of Gait Trainer 3 & RTM600 software can only be used on units with 12-inch display. For any other size display, contact Biodex Customer Service.

Gait Trainer 3

950-400 Gait Trainer 3, 115V – Includes Support Bar

950-402 Gait Trainer 3, 115V – Includes Extended Handrails

950-406 Gait Trainer 3, 115V – Includes Geriatric/Pediatric Handrails

RTM600 Treadmill

950-420 RTM600 Treadmill 115V – Includes Support Bar

950-422 RTM600 Treadmill 115V – Includes Extended Handrails

950-426 RTM600 Treadmill 115V – Includes Geriatric/Pediatric Handrails



Need Help?

Contact Biodex Software Support

1-800-224-6339 ext 2120, or

softwaresupport@biodex.com

Download Instructions – Software 2.02

For Gait Trainer™ 3 & RTM600

On the Biodes website, Physical Medicine section, navigate to the ‘**Software Updates**’ page under ‘**Products**’ in the left navigation bar, and select either Gait Trainer 3 or RTM600 v2.02 Software Upgrade. This will bring up the page with a “**download**” button (See Figure 1; screen will look similar for RTM600) and follow the steps detailed below.



Figure 1. Gait Trainer 3 Software Download screen

How to Download:


NOTE: Screenshots used in this document will match if Internet Explorer 10 and/or 11 is used. Other browsers will look slightly different from these screenshots.

1. Click on the “DOWNLOAD” button to download the software.
2. A “File Download” dialog box will appear.




3. Once the download is complete, open the folder or go to the Downloads folder on your computer.


The GaitTrainer_RTM600_Vers_2.02.zip download has completed.

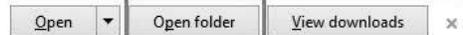


4. Select the file that was just downloaded. (Do **NOT** extract yet.)
Choose the **COPY** option.

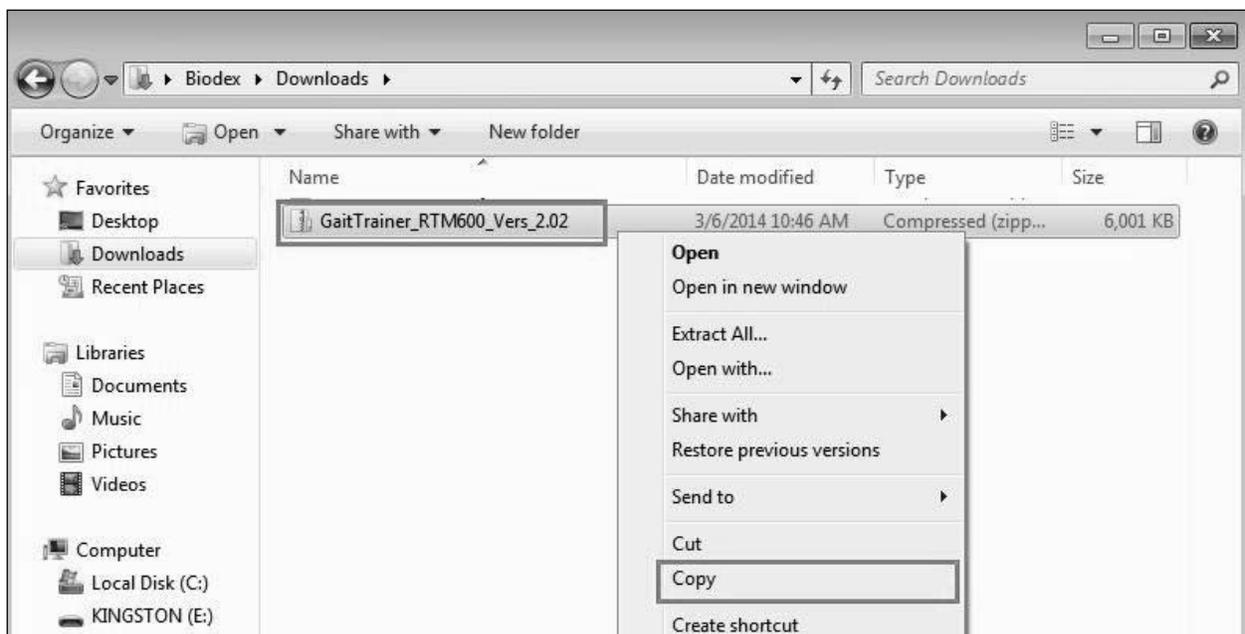

5. Insert Flash Drive and take note of what drive letter is assigned to it.
(In this example we are using the “E” Drive.)
The location may come up as Drive “D”, “E”, “F”, “G”, etc.
6. Open My Computer and Select the Flash Drive.

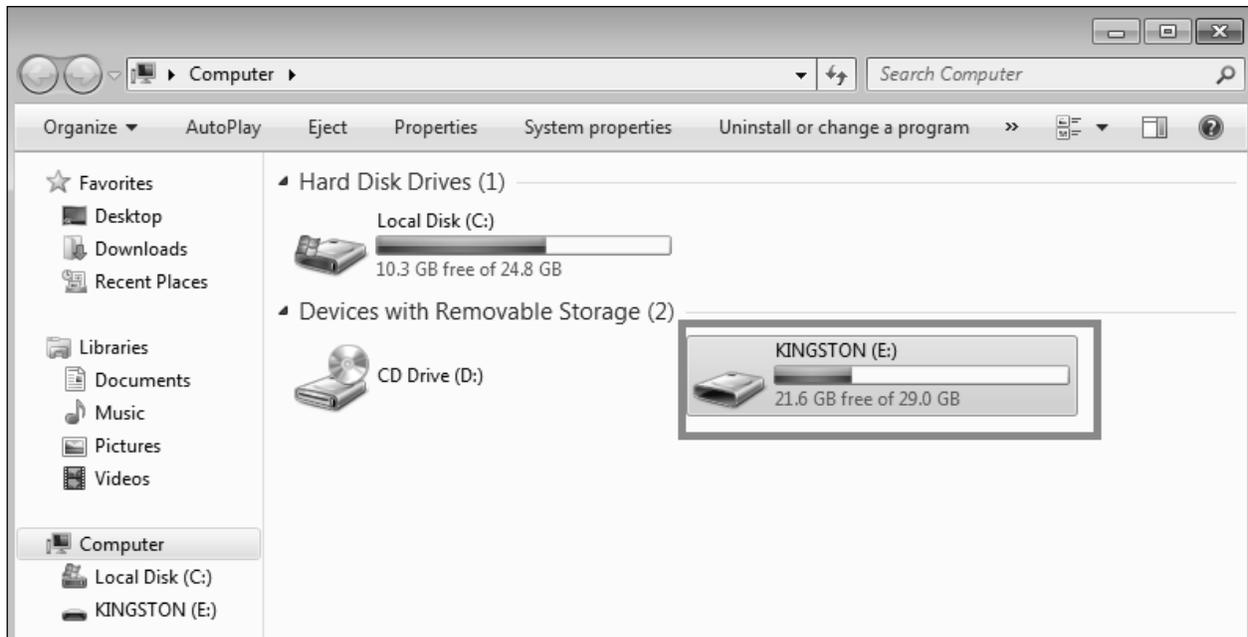


7. Place mouse in center of screen.
Select the **PASTE** option.
Zipped File is copied to Flash Drive.

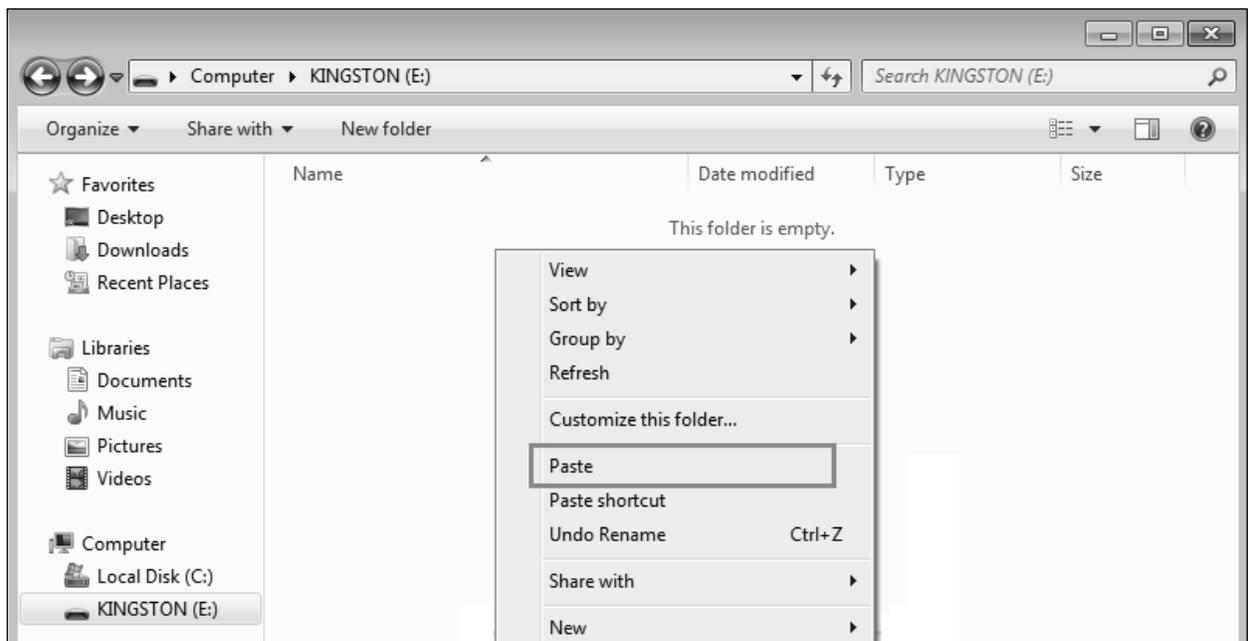


NOTE: It is important to extract the files on the Flash Drive.


8. Select the zip file.

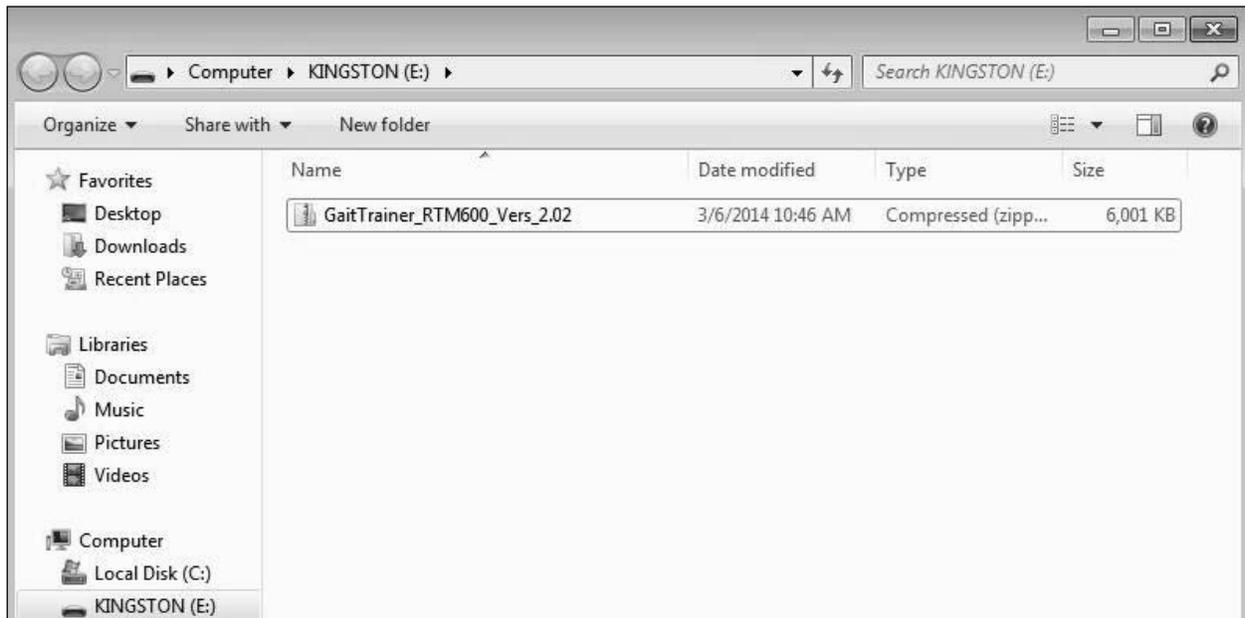


9. Select **EXTRACT ALL FILES**.

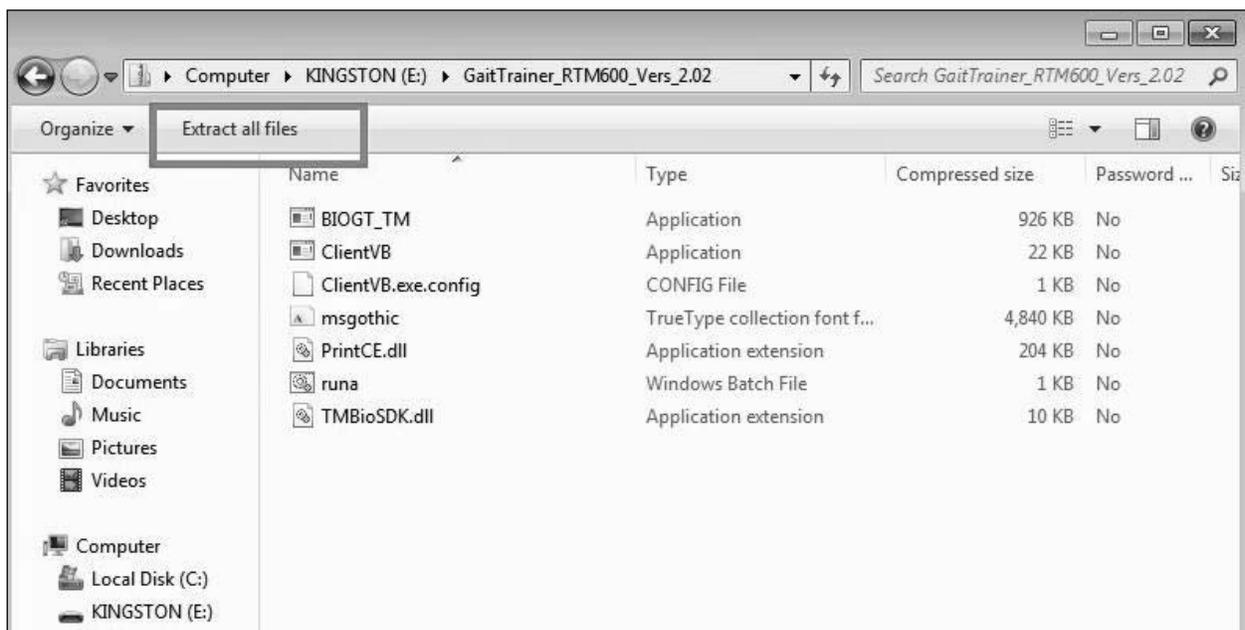

10. Type E:\; Select EXTRACT.

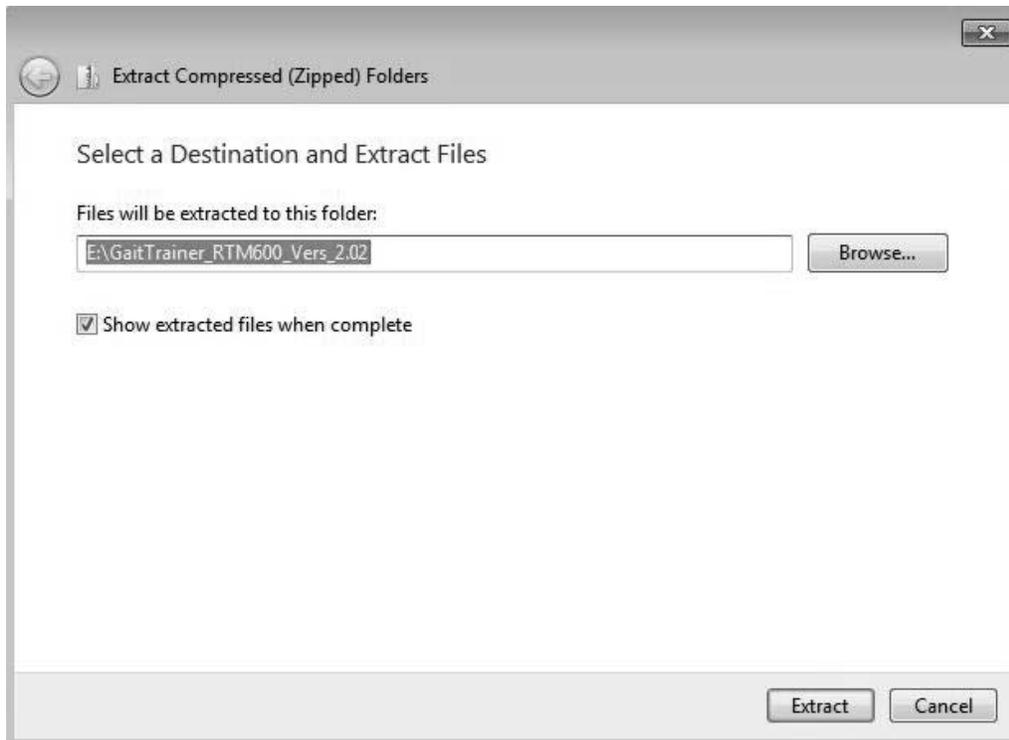


Make sure you type the drive letter (E), a colon (:), and a backslash (\)
E:\ refers to the ROOT of the flash drive.

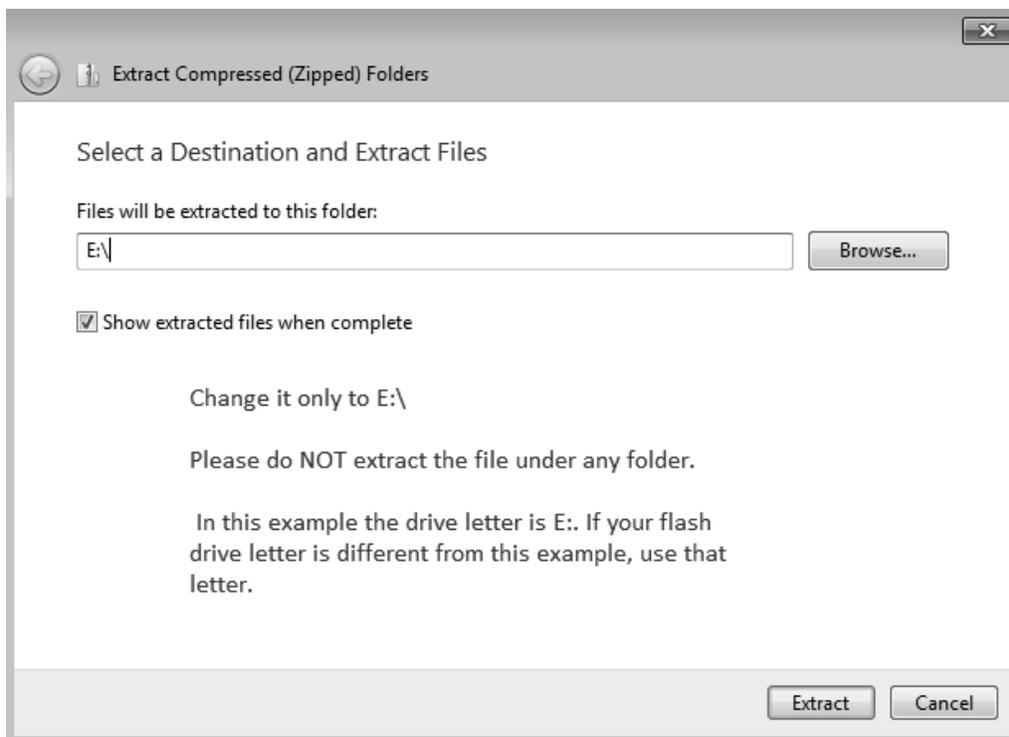


The files will be extracted to the root of the flash drive. It should look like the following screenshot:

IMPORTANT: System will not recognize the updates if they are NOT in the root of the Flash Drive.

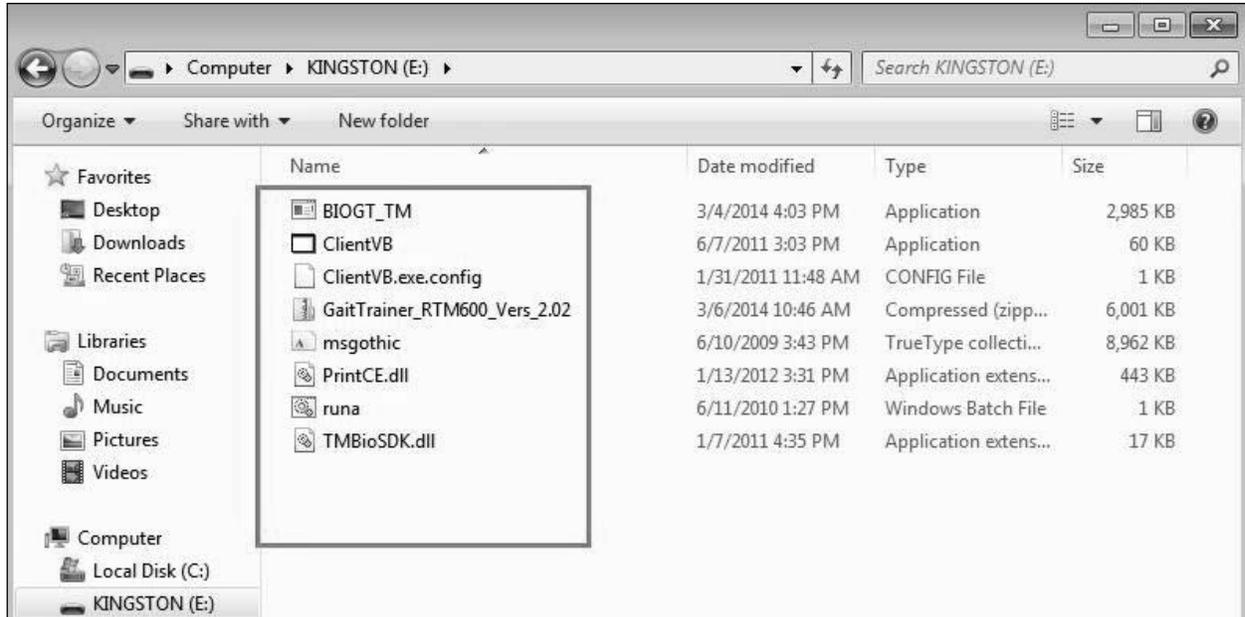

11. Copy is now completed.
12. Close any open windows.
13. Select icon to Safely Remove Hardware (located in lower right hand corner of Desktop).
14. Choose to remove Flash Drive.

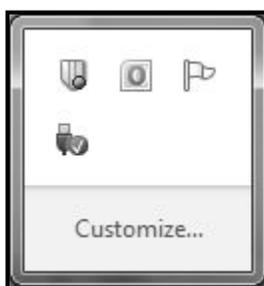

15. Plug in the USB Flash Drive to the display. The unit will automatically detect the software update and display a message on the screen. You will be prompted for automatic updating of software.

BIODEX
Biodex Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (toll free 631-924-9000), Fax: 631-924-9338, Email: info@biodex.com, www.biodex.com