
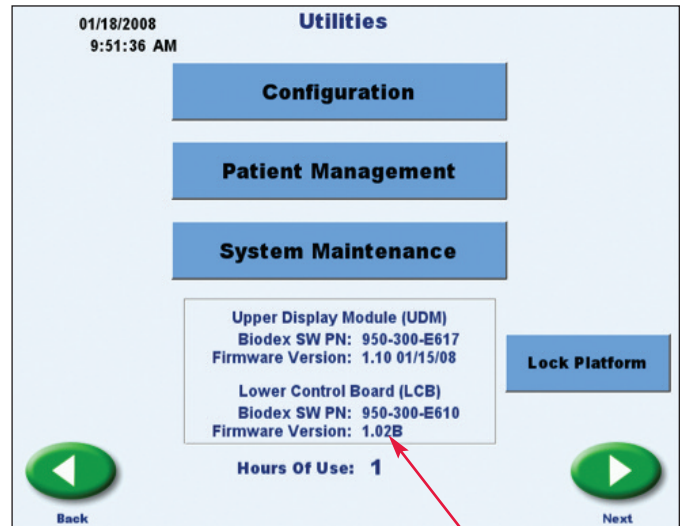


Instructions for Software

VERIFICATION

FOR BALANCE SYSTEM™ SD AND BIOSWAY™ WITH 8" DISPLAY FOR UPGRADE TO 12" DISPLAY

Verify Lower Board Version

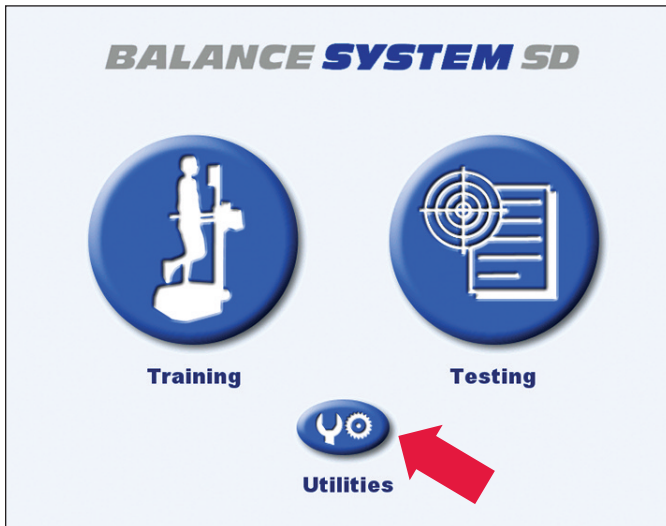

Sample shows Lower Board Version 1.02B

To verify the current Lower Display Board version installed on your present system:

1. Select <Utilities> button on Home screen.
2. Verify Lower Control Board (LCB) version, last line.

Applies to Models: BALANCE SYSTEM SD & BIOSWAY

950-300 Balance System SD
950-330 BioSway

NOTE: Versions 1.02B and below require a service visit and the installation of a new strain gauge board 950-300-E810.

NOTE: Versions 1.03 and above can be upgraded successfully by simply installing display kit 950-440-K900.



Need Help?

Contact Biodes Service Support:

Phone: 1-631-924-9000, ext. 2120

Fax: 1-631-924-8355

Email: servicesupport@biodes.com

BIODEX

Biodes Medical Systems, Inc.

20 Ramsey Road, Shirley, New York, 11967-4704, Tel: 800-224-6339 (Int'l 631-924-9000), Fax: 631-924-9338, Email: info@biodes.com, www.biodes.com